


HANDYSTRAP® FITTING INSTRUCTIONS



1

1. Attach buckle behind neck with shell against chest.

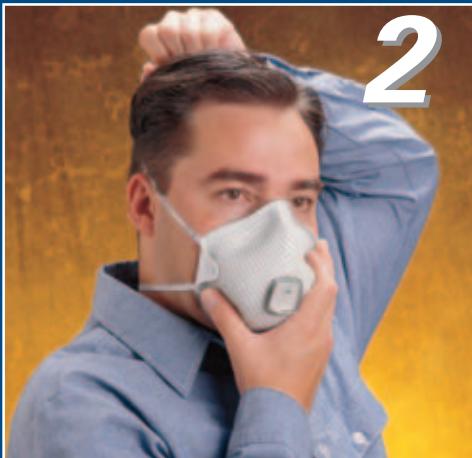

2

2. a) Fit mask to face and pull top of strap to crown of head. Flange must be uncreased inside respirator.
b) Always use top and bottom part of strap. Adjust tension for a comfortable fit.



3

3. Each time, before entering a contaminated area, perform a user seal check. Cover front of respirator by cupping both hands. INHALE SHARPLY. A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust strap by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.



4

4. To hang around neck:
 - a) Undo buckle.
 - b) Allow strap to fall around neck.
 - c) Pull mask down to chest.

OR

 - a) Pull top strap forward away from head.
 - b) Adjust tension to allow mask to hang freely.